# ERGOSWITCH

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### ASSEMBLY AND OPERATING GUIDE

**innOralis**

Tel. +33 (0)3 84 55 89 92 / [contact@innoralis.fr](mailto:contact@innoralis.fr)

18 rue Albert Camus – 90000 Belfort – France

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
Following the installation and operating instructions will increase the life of your ErgoSwitch.

Should you encounter any problems during assembly or use, please do not hesitate to contact us.

## ERGO SWITCH INSTALLATION


### ❖ ATTACHING ON THE CHAIR

**1 – Content of the set of screws** (for a complete pack, people who already have an Aeroshield, refer to point 4-Attaching the female clip to the hose)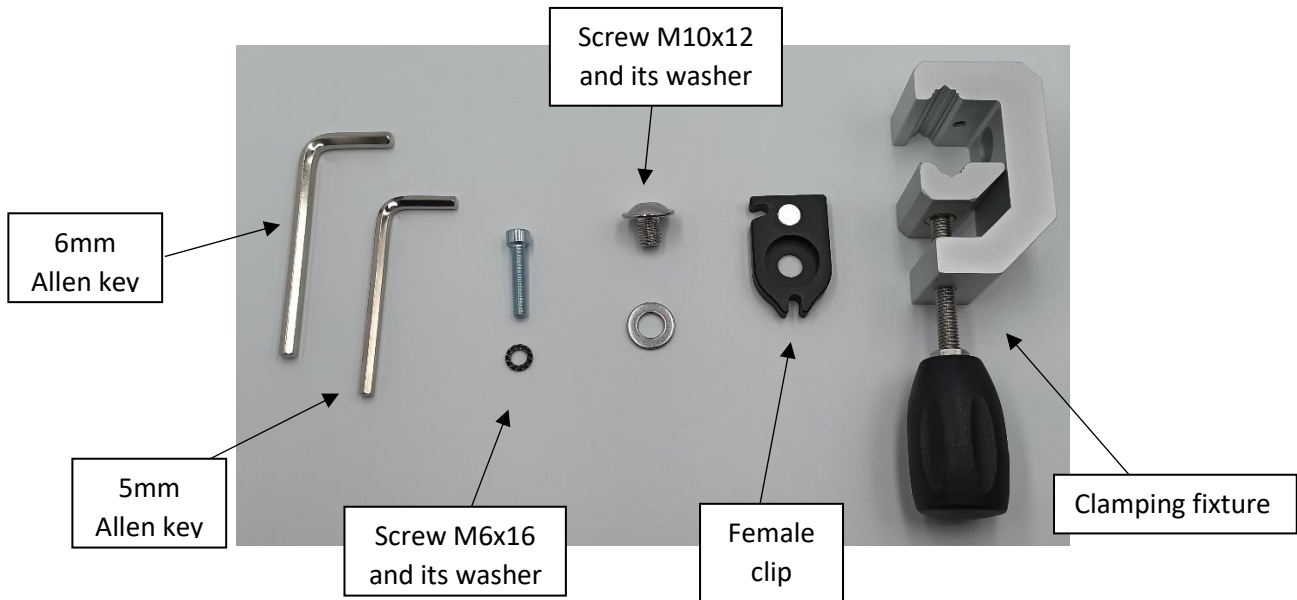

### 2 – Attaching the hose to the clamping fixture

Attach the hose to the clamping fixture with the M6x16 screw with its locking washer on the inside of the clamping fixture and with the double-range washer between the hose and the clamping fixture. Tighten the screw with the supplied 5mm Allen key.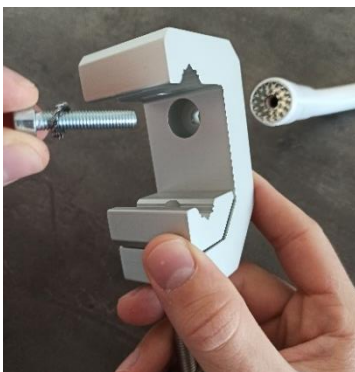

Note: The hoses come with the double-range washer already mounted on the hose.

If, however, it is no longer assembled, mount it again on the hose before attaching on the clamping fixture.

### 3 –Attaching the clamping fixture to the chair

Attach the clamping fixture to the care chair. The clamping fixture adapts to both rectangular and cylindrical sections.


#### WARNING

- Use on a flat surface: Make sure to place the clamping fixture to the bottom of its recess

- Use on a cylindrical surface: Make sure to use the V-notch to secure the desired tube or arm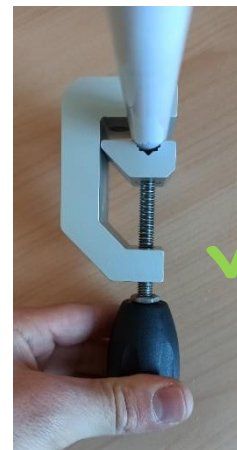

If the clamping fixture is unscrewed, once the tightening screw has come to a stop, do not insist on to not break the tightening screw.

Misuse of the clamping fixture can cause the break of the screw.

This kind of breakage is not guaranteed.

However, a replacement screw can be provided so that you can continue to use your Aeroshield.

Please contact us at [contact@innoralis.fr](mailto:contact@innoralis.fr) if this kind of incident happens.

#### 4 – Attaching the female clip to the flex arm

Put the M10x12 screw in the housing of the female clip then put the washer on the rear face of the female clip, around the emerging M10x12 screw (the washer must be between the female clip and the flexible).

Attach the female clip to the hose using the 6mm Allen key (use the long side to screw quickly then take the short side to tighten firmly).

#### WARNING


The flex arm should be used mainly in bending and should keep large curvatures. Use in twists or with sharp angles can damage the flex arm and shorten its lifespan.

#### 5 – Attaching the accessories to the flex arm


All our accessories are sold with the male clip already mounted.

**N.B:** If, however, you already have Aeroshield mounting plates that you wish to adapt to ErgoSwitch, do not hesitate to contact us at [contact@innoralis.fr](mailto:contact@innoralis.fr) or at 03.84.55.89.92 to present our adaptation solution to you.

Assemble the accessories by first presenting the fixing screw in the axis of the central screw and threading the clip to the bottom of the groove.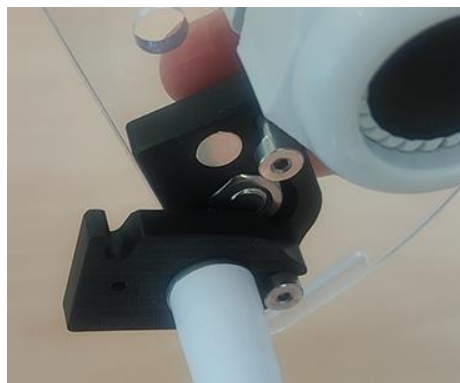

Then, by simply rotating, put the second eccentric screw in the groove: the fixing magnet blocks the clips in the final position.

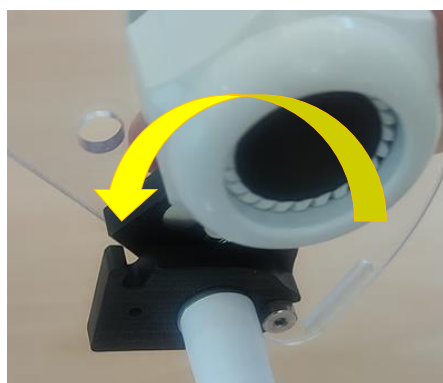### 5.1 - Smartphone support

The InnOralis smartphone holder is compatible with all smartphones.


In order to comply with the hygienic measures of the practice, if it is your personal phone, we recommend that you use a protective cover suitable for your smartphone model to clean with a wipe between each use.

Otherwise, if it is a smartphone dedicated to the practice, you can do without this protective cover. Make sure, however, that your smartphone is resistant to disinfection cleaning (waterproof).

### 5.2 - Aeroshield

#### 5.2.1 - Preparation

Remove the adapter from the support plate to remove the two protective plastics (**one on each side**).

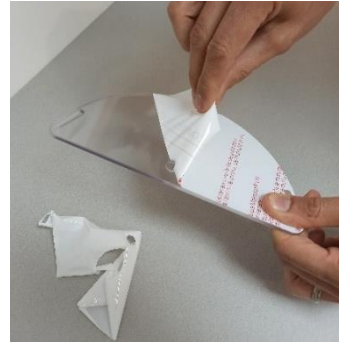

Reassemble the adapter on the support plate.

### 5.2.2 – Attaching the bubble on the support plate

Remove the two protective plastics from the bubble (**one on each side**).

In order to easily and quickly clip the bubble onto the support plate, first position the central clip, then press the sides, attach the 2 side clips.

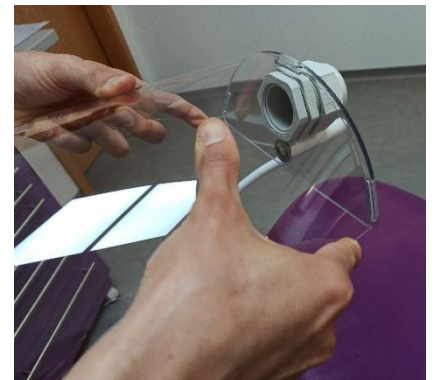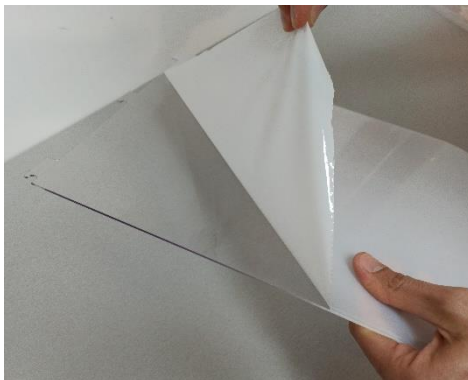### 5.2.3 – Positioning the Aeroshield

#### 5.2.3.1 – Positioning the axial version

In order to optimize your gestures, protection and aerosol capture, we recommend using the axial version with an angle of 45 degrees as shown below.

### 5.2.3.2 – Positioning the lateral version

In order to optimize your gestures, protection and aerosol capture, we recommend using the lateral version with the aspiration adapter upwards and oriented as shown below.


### ❖ USING AN ASPIRATION

#### 1 – Surgical aspiration

Remove the adapter to remove the stopper cap.

Bring your surgical aspiration inside the adapter and then screw the nut to hold your aspiration. The adapter allows the connection of pipes from 18 to 32mm in diameter.
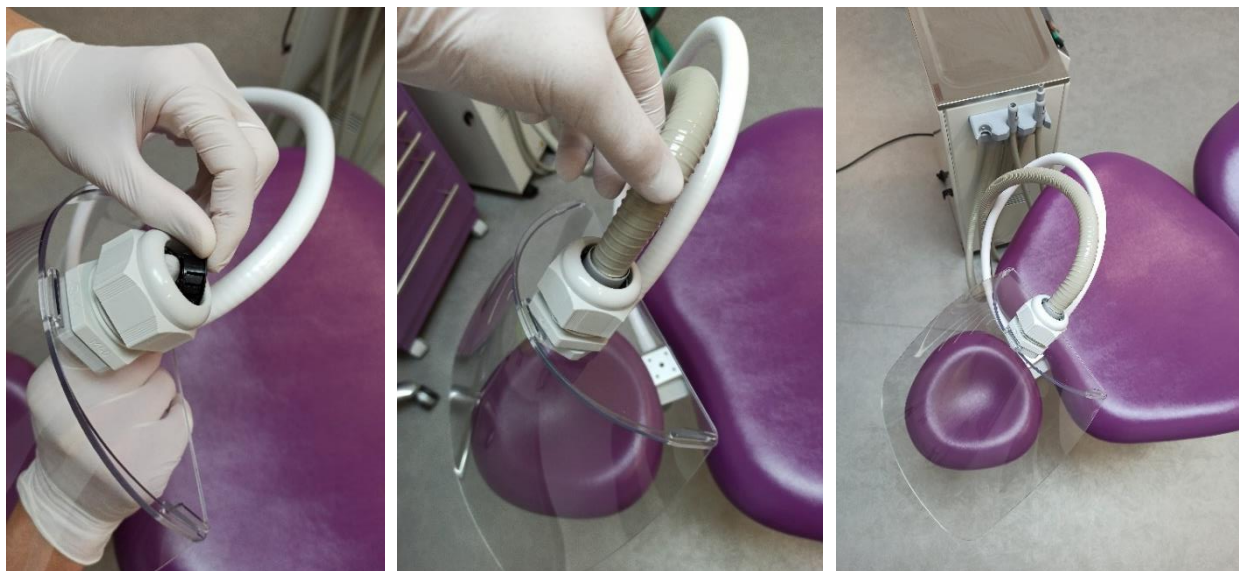

## 2 – Clean room aspiration

If you want to use a dedicated clean room vacuum cleaner, depending on the diameter of your suction hose, you may need an adaptive hose. We can get you an adaptive hose of 1m50 in length, please contact us at [contact@innoralis.fr](mailto:contact@innoralis.fr).

Connect the adaptive hose to your vacuum cleaner either by threading the adaptive hose into your vacuum hose (1), or by using the rigid elbow of your vacuum cleaner (2).

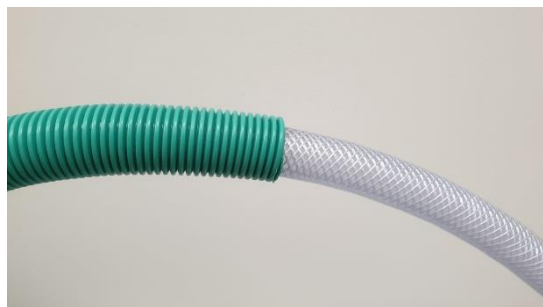*Solution 1*

*Solution 2*

Then connect this adaptive hose into the Aeroshield adapter as explained in the previous procedure.

### ❖ USING AN OPERATING FIELD

With the Aeroshield kit, 4 clamps are supplied to attach an operating field to the Aeroshield and thus limit the projection of aerosols.

These clamps are made of stainless steel and are washable to facilitate decontamination and will therefore be resistant over time.

**⚠ WARNING:** Although these clamps are made of stainless steel, be careful not to soak them for too long and dry them well, especially on the contact areas, to limit the appearance of oxidation stains.

### AEROSHIELD CLEANING

The Aeroshield Kit can be cleaned in several ways: **COLD CLEANING ONLY**

- Cleaning with non-abrasive wipe (wipe like paper towels must be prohibited, it can cause micro-scratches)
- Cleaning by alcohol spray and drying by special glass microfiber wipe or soft cloth
- Cleaning by soaking in disinfection bath

### ⚠ WARNING

Don't clean in an autoclave cycle. InnOralis does not assume any liability for any damage caused by hot cleaning of the Aeroshield.

Remove the pieces of resin or amalgams from the bubble before wiping it with the disinfection wipe to avoid scratches.

### ⚠ IMPORTANT

**It is recommended to clean your Aeroshield after disassembling the protective bubble** and to dry the parts thoroughly between each use.

Cleaning your mounted Aeroshield and not drying it immediately will cause the stagnation of microdroplets of corrosive substance in the hollow areas, chemically attacking the plastic that can damage or even break the parts of your Aeroshield.

If all of these procedures are followed, your Aeroshield kit will extend its lifespan: there are no limited period of time for use of the kit.

The bubbles can deteriorate over time. They are available for restocking if you wish to replace them. Namely, being made of PETG, they are recyclable!

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